bosotherm basic

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ⓁⒹⒺ User Instructions

ⓁⒹⒺ Mode d'emploi

ⓁⒹⒺ Manual del Usuario

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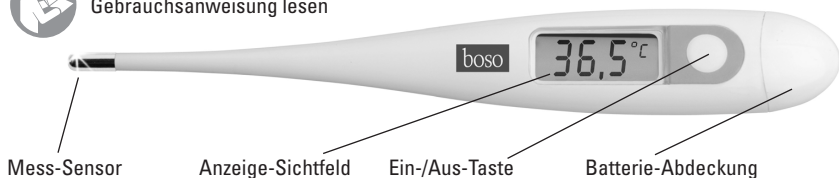
 **DE** Symbole auf dem Thermometer /
  **EN** Symbols on the thermometer /
  **FR** Symboles sur le thermomètre /
  **ES** Símbolos en el termómetro /
  **IT** Simboli sullo termometro

Symbol / Symbol / Symbole / Símbolo / Simbolo	Funktion – Bedeutung / Function/meaning Fonction/signification / Función/significado Funzione/Significato
	Ein-/Aus-Taste ON/OFF-button Interrupteur marche/arrêt Botón ON/OFF Copercio
	Keine weitere Messung möglich. Batterien ersetzen. No further reading possible. Replace battery. Une mesure n'est plus possible. Remplacer la batterie. No son posibles más mediciones. Sustituir las baterías. Impossibile continuare la misurazione. Sostituire le batterie.
	Gerät vom Typ BF Type BF equipment Appareil de type BF Aparato del tipo BF Apparecchio di tipo BF
	Lotnummer Lot Number Numéro de Lot Número de Lot Numero di Lot
	Hersteller Manufacturer Fabricant Fabricante Produttore
	Gebrauchsanweisung lesen Read instructions for use Lire le mode d'emploi Leer las instrucciones de uso Leggere le istruzioni per l'uso
	Gerät darf nicht über den Hausmüll entsorgt werden. Device must not be discarded with household waste. L'appareil ne doit pas être jeté à la poubelle. El aparato no se debe eliminar en la basura doméstica. L'apparecchio non deve essere smaltito con i rifiuti urbani generici.
Lo °C	Temperatur unter 32 °C Temperature under 32 °C Température inférieure à 32 °C Temperatura inferior a 32 °C Temperatura inferiore a 32 °C
H °C	Temperatur über 42.9 °C Temperature over 42.9 °C Température supérieure à 42.9 °C Temperatura superior a 42.9 °C Temperatura superiore a 42.9 °C
CE 0124	Gerät ist konform mit der europäischen Medizinprodukterichtlinie. Device complies with the European Medical Devices Directive. L'appareil est conforme à la directive européenne « Dispositifs médicaux ». El aparato cumple la Directiva sobre Productos Sanitarios europea. L'apparecchio è conforme alla direttiva europea sui dispositivi medici.

bosotherm basic

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Mess-Sensor

Anzeigefeld

Ein-/Aus-Taste

Batterie-Abdeckung

CE 0124

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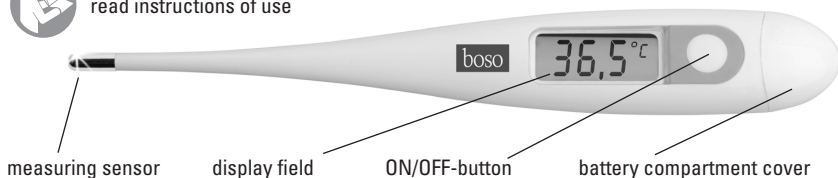
CE 0124

Measuring methods

Before each use of the thermometer, make sure that it is not damaged.

Oral (60 ± 10 seconds)
normal temp.:
35,7°C – 37,2°C

Rectal (60 ± 10 seconds)
normal temp.:
36,2°C – 37,7°C

Axillary (100 ± 20 seconds)
normal temp.:
35,2°C – 36,7°C

MEASURING METHODS

• In the mouth (oral)

Put the thermometer underneath your tongue. The instrument must have good contact with the tongue. Close the mouth and breath calmly through your nose in order not to influence the measurement by the inhaled air. Minimum measuring time: 60 seconds.

• In the anus (rectal)

This is the most reliable method and the most appropriate for babies and children. Carefully immerse the tip of the thermometer 2-3 cm into the anus. Minimum measuring time: 60 seconds.

• Underneath the arm (axillary)

Under medical aspects this measuring method will lead to inaccurate results. Therefore it is not appropriate to get exact measurements. Minimum measuring time: 100 seconds.

IMPORTANT SAFETY INSTRUCTIONS

- Before use, the device should be allowed to adjust to ambient temperature (i.e., room temperature). If there is a change in the ambient temperature, for example, due to a temperature difference between the places where the device is stored and where it is used, you should wait at least 30 minutes before use so that the device can adapt to a stable ambient temperature (place of use)!
- The thermometer is appropriate for the measurement of the body temperature only!
- This device can be used at home and in hospital.
- The patient can be the user.
- Users with special attributes, such as disabilities, advanced age, or children, who may require assistance from another person to enable correct use of the medical device must not use the device unsupervised.
- Do not interrupt the measurement before the signalling tone [beep] indicates that the minimum measurement time is completed.
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from tremor and shock!
- Do not expose thermometers to direct sunlight and NEVER boil them!
- Only water or the disinfectants listed in the section «cleaning and disinfecting» are to be used for cleaning. The thermometer must be intact when it is immersed in liquid disinfectant.
- The thermometer has a long-life battery. For disposal, please comply with local regulations for special waste..
- Regular checks on the accuracy of the measurement system should be carried out in accordance with the valid directives of the individual countries, governing this issue.
- Wireless communication devices, such as home networking devices, mobile phones, cordless phones and their base stations, walkie-talkies can affect this thermometer. Therefore, a minimum distance of 11 feet should be kept from such devices.
- ⚠ WARNING: The measurement result given by this device is not a diagnosis! Do not rely on the measurement result only kept from such devices.

PURPOSE

Determination of body temperature of persons

TURNING ON THE THERMOMETER

The device must be installed and put into operation in accordance with the instructions in this manual.

To get help concerning usage or maintenance please contact your dealer or the manufacturer.

An unexpected operating condition or an incidence which has worsened the health condition or could have worsened the health condition should be reported to the manufacturer immediately.

To turn on the thermometer, push the ON/OFF-button; a short signalling tone [beep] indicates «thermometer ON.» A so-called LCD-test automatically takes place to check the display. Then a «Lo» and a flashing «°C» appear on the upper right of the display, if the ambient temperature is less than 32°C. Now the thermometer is ready for use.

USING THE THERMOMETER

During a measurement, the temperature is displayed continuously and the «°C» symbol flashes. If the signalling tone sounds (beep-beep-beep-beep 10 times) and the «°C» no longer flashes the measurement is completed (the measured increase in temperature is now less than 0,1 °C in 15 seconds).

To extend the lifetime of the battery, turn off the thermometer after usage by pressing the ON/OFF-button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

MEMORY OF MEASURED VALUES

When turning the thermometer on, the temperature of the last measurement will be displayed. After 3 seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

CLEANING AND DISINFECTING

For cleaning of the instrument please use isopropyl alcohol (70%). For disinfecting wipe the instrument completely with Antifect Liquid (please obey 5 min. work-in-time).

CHANGE OF BATTERY

As soon as the symbol „□“ appears on the display, the battery is weary and a new battery has to be inserted. Open the cover of the battery compartment and insert the new battery with „+“symbol facing upwards.

If the device is not used for an extended period (4 weeks or more), remove the battery in order to prevent any damage that may be caused by leakage.

TECHNICAL SPECIFICATIONS

Maximum thermometer

Measuring range: 32.0°C to 42.9°C

Temp. < 32.0°C: display «Lo» for low (too low)

Temp. > 42.9°C: display «H» for high (too high)

Accuracy: ± 0.1°C between 35,5°C and 42,9°C

Operating conditions: environmental temperature 5°C to 35°C, air pressure 700 – 1060 hPa, relative humidity 40 to 80%

Liquid crystal display (LCD) with three digits. Smallest unit of display: 0.1°C

Signalling tone sounds when thermometer is ready for use and when measurement is completed.

Memory: automatic memory of the last measured value.

Storage conditions: environmental temperature

-25°C to +55°C, relative humidity 15% - 95%

Battery: 1,5 V, IEC-Type LR/SR41

1,000 measurement cycles (depending on frequency of use).

Norm: ISO 80601-2-56, Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement

Protection against solid objects and water: IP 67 (in accordance with IEC 60529)

Expected operational lifetime of the equipment: 10 years

WARRANTY

This unit is under **warranty for 2 years** from the date of purchase.

Please submit the receipt, invoice or delivery note when claiming for warranty.

For both, warranty and repair, please send the instrument carefully packed and postage paid either to your authorized dealer or directly to

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72417 Jungingen – Germany

Warranty and repair work must be undertaken by trained and authorised personnel. Do not modify this equipment without authorization of the manufacturer.

CARE FOR THE ENVIRONMENT

Used batteries and defective electronic devices do not belong into the household waste. 

They can be deposited at special centres. Please contact your local authorities for information.

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Bosch + Sohn GmbH und Co. KG