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DE

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Instructions for use

EN

Please read the instructions for use carefully because they contain important information about the use of your KONTICUR® products.


Urostomy bed bag

Intended use: The KONTICUR® urostomy bed bag is designed for people with an artificial bladder outlet. The urostomy bed bag can be used for draining, holding and collecting urine at night. The product is intended for single use.

The KONTICUR® urostomy bed bag KUBB2000 can be combined with all STOMOCUR® urostomy bags from FORLIFE.

General preparation: Wash your hands carefully before and after each use.

Note: The urostomy bed bag (Fig. 1) can be combined with a bag hanger (e.g. KONTICUR® KBA, Fig. 2) for hanging (available as an accessory product).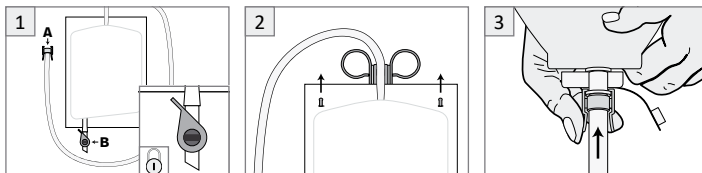


Connecting to a urostomy bag

Before connecting the urostomy bag (Fig. 3), make sure that the discharge valve (Fig. 1, B) of the urostomy bed bag is closed (Fig. 1, small picture).

To attach the urostomy bed bag, hang the two spikes of the KBA in the perforations provided, on the top right and left of the bag (Fig. 2). Place the urostomy bed bag at or below bladder level.

Connect the urostomy bed bag to the urostomy bag. To do this, push the adapter (Fig. 1, A) of the bag over the outlet of the urostomy bag until you hear a click (Fig. 3). The urostomy bed bag is now securely connected.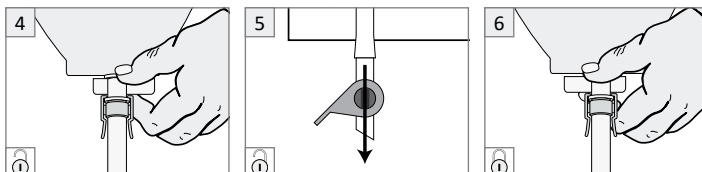

Emptying

Open the urostomy bag to allow the urine to drain into the urostomy bed bag. To do this, turn the twist cap of the urostomy bag 180 degrees (Fig. 4).

Empty the urostomy bed bag by turning the lever anticlockwise so that the lever points downwards (see Fig. 5).

Seal the urostomy bag first, before removing the bed bag. To do this, turn the twist cap of the urostomy bag back to its original position (Fig. 6). To remove the urostomy bed bag, press on the two ends (Fig. 3) of the adapter (Fig. 1, A) and pull it off in a downwards direction.

Warnings

- This product is intended for single use, further use of this disposable product may pose a potential risk to the user.
- Please use the KONTICUR® product specified in these instructions for use for the purpose stated in each case (see intended use).
- Do not use these products for other purposes.
- Please empty the bag before disposing of it properly. KONTICUR® products must not be disposed of in the toilet.
- If you have any questions or problems with use, please contact your specialist doctor or your ostomy therapist.
- In case of serious injury (incident) related to the use of this medical device, please contact your local distributor or the manufacturer, as well as your local competent authority.

Storage conditions

- Store in a cool, dry place away from light.

(DE) Distribution, (EN) Distribution

Oakmed Limited

Unit B, The IO Centre
Barn Way
Lodge Farm
Industrial Estate
Northampton NN5 7UW
United Kingdom
+44 (0) 1604 586 529
info@oakmed.co.uk
www.oakmed.co.uk