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# Directions for use for TRACHEOCUR® STOMA-OIL

## TRACHEOCUR® STOMA OIL

Item No.: T123000, 50 ml

The directions for use are intended to inform the doctor, nursing staff and patients/users about correct handling. Please read carefully before use.

### INTENDED PURPOSE

TRACHEOCUR® STOMA OIL increases the slippage of the tracheal cannula during insertion. It is also used for skincare around the tracheostoma.

### DESCRIPTION

TRACHEOCUR® STOMA OIL is a neutral lubrication oil that is made from medium chain triglycerides.

### APPLICATION

Before insertion, a few drops of TRACHEOCUR® STOMA OIL are applied to a compress and then rubbed onto the cannula and the edges of the tracheostoma.

TRACHEOCUR® STOMA OIL is supplied in a dropper bottle with a screw top to aid dosing.

### SAFETY INFORMATION

TRACHEOCUR® STOMA OIL is packaged in an unsterile condition and has a best before date. It must not be used longer than for one month from the time the bottle is opened.

### After eye contact:

Rinse opened eyes with plenty of running water.

### After swallowing:

Have the person drink plenty of water. If discomfort persists, seek medical attention.

**If you have any problems with the product, please contact our customer service:**

**0800 8722436**

### DISPOSAL

If the patient is cared for at home, it is to be assumed that they are not infectious. In this case, disposal with the normal domestic waste is possible. If the patient is being treated in a hospital by medically trained professionals, the disposal of (poss. infectious) materials used on the patient is regulated by law. In this way, TRACHEOCUR® STOMA OIL is disposed of in the surgery or hospital.

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BMV® Bender Medical Vertrieb GmbH