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INSTRUCTION FOR USE

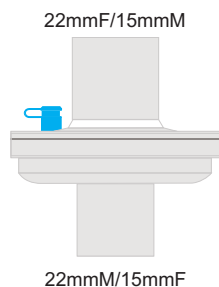
NOVO FI Breathing Filter S

Bacterial/Viral Filter


REF N16410010

IFU_Rev. 01_202209

Bestandteile Components

Tidalvolumen / Tidal Volume:	150-1500 ml
Durchflusswiderstand bei / Resistance to Flow at:	30 l/min: ≤ 0.8 cmH ₂ O 60 l/min: ≤ 1.7 cmH ₂ O 90 l/min: ≤ 2.7 cmH ₂ O
Totraum / Dead Space:	33 ml
Filter-Art: Mode of Filtration:	Elektrostatisch Electrostatic
Bakterielle Filtrationseffizienz: Bacterial Filtration Efficiency:	99,999 %
Virale Filtrationseffizienz: Viral Filtration Efficiency:	99,99 %
Anschlüsse / Connectors:	22F-15M/22M-15F

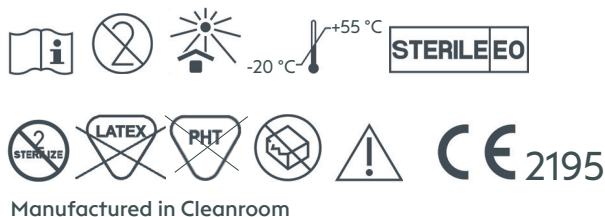Patientenzielgruppe
Patient target group

Beatmete Patienten / Anästhesie
Ventilated patients / anesthesia

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EN | INSTRUCTION FOR USE

Bacterial/Viral filter prevents bacteria and viruses from reaching the patient's respiratory tract. For use in adolescents and adults.

1. USEFUL LIFE

For single use only, 24 hours usage time recommended.

2. NOTICE ON REPORTING REQUIREMENTS

Please inform the manufacturer and the competent authority of the Member State in the event of any serious incident that has occurred in relation to the device.

3. Application by healthcare professionals or users trained by healthcare professionals.

CONTRAINDICATIONS

The device is used only as directed. Improper use of it will harm the patient.

WARNING NOTICES

- To avoid contamination and soiling, the product should remain packaged until ready to be used.
- Check all system components for obstructions, damage, and foreign matter before installation.
- Using incompatible components may adversely affect the performance of the ventilator.
- If secretions are visible, replace the filter immediately.
- The use of additional components requires particular care and monitoring.
- Monitor filter performance during the use.
- Do not use the Bacterial/Viral HME Breathing Filter in combination with active humidifiers or medication nebulizers.
- The device is sterilized with Ethylene Oxide. Sterile conditions are guaranteed only if the package is not damaged.

Novo Klinik-Service GmbH
Technologiepark West – Haus 15 |
Zum Frenser Feld 1
50127 Bergheim | Germany

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