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EN | Instructions for use

Tracheal non-woven dressing for the care of patients with a tracheostomy, used to absorb tracheal secretions and to cushion the area between the neck and the tracheal cannula. Suitable for infants, (young) children, adolescents, and adults.

1. Useful life / replacement frequency Change when soiled / defective / according to hygiene guidelines. The interval between changing the tracheal dressing depends on the level of saturation of the dressing. Maximum usage duration is 24 hours.

2. Notice on reporting requirements Please inform the manufacturer and the competent authority of the Member State in the event of any serious incident that has occurred in relation to the device.

3. Contraindications

Tracheal dressings must not be used for ulcers resulting from infections such as tuberculosis, syphilis, and fungal infections.

4. Warning notices

- Clean the tracheostomy before using the product.
- Necroses should be removed before using the product.
- Do not use the product in case of damaged packaging or improper storage; it should be discarded.
- The product is intended for single use. Reuse can lead to infection and/or impairment of functionality.

5. Users

Application by healthcare professionals or users trained by healthcare professionals. Decision on use is made by the treating doctor.

6. Notice for use by laypersons

The product is very easy to use and does not require any special skills or equipment. If complications (such as inflammation, etc.) occur, contact a doctor immediately.

7. Instructions on disposal

The product should be disposed of after use in accordance with legal and local regulations.

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