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**LOT** 210508  
**EXP** 2021-07  
**EXP** 2026-06

**Hersteller | Manufacturer**  
Ningbo Shengyurui Medical Appliances CO., LTD  
No 138, Binhaisi Road, Hangzhou Bay New Zone,  
315336 Ningbo, Zhejiang Province P.R. China

**Importeur | Importer**  
Novo Klinik-Service GmbH  
Technologiepark West - Haus 15  
Zum Frenser Feld 1 | 50127 Bergheim  
Germany  
Tel.+49 2271 99404-0 | Fax +49 2271 99404-100  
info@its-novo.de | its-novo.de

**EU** **REP** **EU** **Repräsentant | EC** **Representative**  
Shanghai International Holding Corp. GmbH (Europe)  
Eiffestrasse 80, 20537 Hamburg  
info@shholdingeu.com

Revison: 01-20210112-IFU-ICVMA03

## Nebulizer set for babies, incl. mask and oxygen sure flow tubing

**REF** IC-VMA03 Quantity: 1 Set **MD** **CE** 1639

### CONTENT:

Nebulizer set for babies, incl. mask and 210 cm oxygen sure flow tubing

### INDICATIONS

Mainly used for the inhalation of medication as aerosol / in the treatment of various diseases of the respiratory system.

### INTENDED USERS

Nursing staff, medical professionals

### APPLICATION DESCRIPTION

- Place prescribed medication in nebulizer jar by unscrewing lid.
- Reattach lid.
- Insert the mask into the nebulizer and connect the nebulizer to the flow source using an O<sub>2</sub> tube.
- Set liter flow as prescribed by physician.
- Place the mask on the patient and secure the mask with the head strap.

### WARNING NOTICES

- For single use. Discard after use. Do not reprocess.
- Do not use if package is open or damaged
- Store in cool and dry place
- Ensure that all connections are secure and air circulates freely through the tubing

### NOTICE ON REPORTING REQUIREMENTS

Please inform the manufacturer, the distributor and the importer in all cases of serious incidents in connection with the product.

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