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2021-07

2026-06

Hersteller | Manufacturer
Ningbo Shengyurui Medical Appliances CO., LTD
No 138, Binhaisi Road, Hangzhou Bay New Zone,
315336 Ningbo, Zhejiang Province P.R. China

EU Repräsentant | EC Representative
Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg
info@shholdingeu.com

Importeur | Importer
Novo Klinik-Service GmbH
Technologiepark West - Haus 15
Zum Frenser Feld 1 | 50127 Bergheim
Germany
Tel. +49 2271 99404-0 | Fax +49 2271 99404-100
info@its-novo.de | its-novo.de

Revison: 01-20210112-IFU-ICVMA02

Nebulizer set for adults, incl. mask and oxygen sure flow tubing

REF IC-VMA02 Quantity: 1 Set **MD** 1639

CONTENT:
Nebulizer set for adults, incl. mask and 210 cm oxygen sure flow tubing

INDICATIONS

Mainly used for the inhalation of medication as aerosol / in the treatment of various diseases of the respiratory system.

INTENDED USERS

Nursing staff, medical professionals

APPLICATION DESCRIPTION

- Place prescribed medication in nebulizer jar by unscrewing lid.
- Reattach lid.
- Insert the mask into the nebulizer and connect the nebulizer to the flow source using an O₂ tube.
- Set liter flow as prescribed by physician.
- Place the mask on the patient and secure the mask with the head strap.

WARNING NOTICES

- For single use. Discard after use. Do not reprocess.
- Do not use if package is open or damaged
- Store in cool and dry place
- Ensure that all connections are secure and air circulates freely through the tubing

NOTICE ON REPORTING REQUIREMENTS

Please inform the manufacturer, the distributor and the importer in all cases of serious incidents in connection with the product.

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