


- DE** Anlegen des unteren Fixierbands zum Abmessen. Anschließendes individuelles Kürzen mit einer Schere.
- EN** Apply the lower fixation strap for measurement, then cut individually with a pair of scissors.
- ES** Aplíquese la correa baja de fijar para medir, despues corte individualmente con Tijeras embotadas.
- IT** Applicare la fascia di fissaggio inferior par la misurazione, quindi tagliare a misura con una forbice smussa.
- FR** Appliquer la bande de fixation inérieure pour verifier la dimension, raccourcir ensuite à la dimension souhaitée avec des ciseaux émoussés.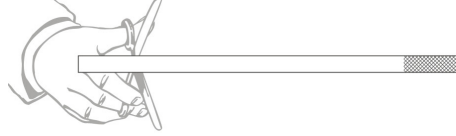


- DE** Einlegen des Urinbeutels in die Fixierlaschen des Klettbandes REF 4896 und Durchziehen des unteren Fixierbands.
- EN** Put the urine bag into the fixation clips of REF 4896 and pass/pull through the lower strap.
- ES** Pone la bolsa de orina a la ciere de fijar del REF 4896 y paso por la correa baja.
- IT** Applicare la sacca per urina nelle clip di fissaggio della fascia rif. 4896 e tirare la fascia di fissaggio inferiore.
- FR** Mettre la poche pour urine dans les pattes de fixation de la ande Velcro REF 4896 et tirer complètement la bande de fixation inférieure.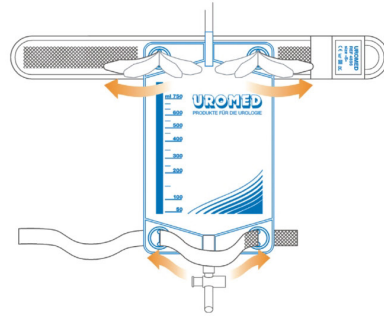

- DE** Anlegen an den Oberschenkel
- EN** Application on the femur
- ES** Aplíquese en el femur
- IT** Applicazione sulla coscia
- FR** Appliquer sur la cuisse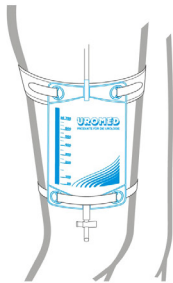

- DE** Anlegen an den Unterschenkel
- EN** Application on the lower leg
- ES** Aplíquese en el muslo
- IT** Applicazione sulla gamba.
- FR** Appliquer sur la jambe

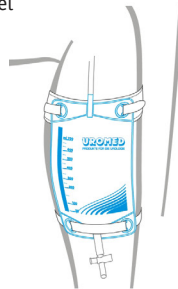**UROMED Klettband-Set**  S  M  L

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# UROMED

- (DE) Klettband-Set  
(EN) Hook-and-Loop-strap Set  
(ES) Kit de cinta de gancho y bucle  
(IT) Kit di nastri per chiusura a strappo  
(FR) Jeu d'attaches auto-agrippantes

**UROMED**  
Kurt Drews KG  
Meessen 7/11  
D-22113 Oststeinbek

**Telefon** +49 (0) 40 / 7 13 007-0  
**Telefax** +49 (0) 40 / 7 13 007-99  
**Internet** [www.uromed.de](http://www.uromed.de)  
**E-Mail** [service@uromed.de](mailto:service@uromed.de)

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## Read all the instructions carefully before use

### Definition of use

Medical Device for fastening of UROMED urine leg bags

### User

Physicians, trained nursing staff or appropriately trained patients/ relatives.

### Patient Group

Adults and children using a urine bag for urine drainage.

### Indication

The UROMED Hook-and-Loop-strap Set is an unsterile medical product for device for fastening ofUROMED urine leg bags on the leg of patients using (bladder/kidney/male external) catheters to collect urine.

### Contraindication

Do not use the device when suffering from circulatory and/or sensory disorders in the legs or known thrombosis of the leg veins. Do not continue to use the device if an allergy or hypersensitivity occurs.

### Clinical Use

To maintain mobility when using a urine leg bag and improvement of quality of life.

### Application

Select a suitable size using the size table. The lower / thinner Hook-and-Loop-strap can be shortened with scissors if necessary. To measure the correct length, apply the tape once before.

Place the Hook-and-Loop-strap on the thigh / lower leg with tabs facing outwards.

Thread the tabs through the tab openings on the urine bag.

In order to fix the urine bag even better, the

lower part of the bag can be fixed by the pre-punched tab openings with a second strip.

Before use, have yourself instructed in the use of the device by qualified personnel.

### Pre-treatment and preparation

Check the device for damage after each wash. Do not use a damaged product.

Do not use the device after the expiry date.

### ⚠ Warnings

Do not wear the product on injured skin.

The user should select a suitable size. This prevents vascular compression and secondary health problems.

Do not shorten the Hook-and-Loop-strap on the leg – risk of injury!

Please consult your doctor if you are not sure whether the product is suitable for you.

Report any serious incident that has occurred in relation with the device to UROMED Kurt Drews KG and the competent authority in your country.

### Storage and handling conditions

Store the device at room temperature, dry, dust-free and protected from light.

### Disposal

After use, the medical device may present a biological hazard. After use, dispose the device in accordance with local regulations and safety rules (e.g. with household or hospital waste).

### Technical description and performance features

The UROMED Hook-and-Loop-strap Set can be combined with all UROMED urine leg bags up to a maximum filling quantity of 750 ml.

REF	Sizes		
4896.01	Small	»S«	23-33 cm
4896.02	Medium	»M«	33-51 cm
4896.03	Large	»L«	51-68 cm

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